**CITEST™**

**COVID-19 Antigen Rapid Test (Swab)**

Number: 14666101 Date: 2022-01-26

A rapid test for the qualitative detection of SARS-CoV-2 Nucleocapsid Protein antigens present in nasal swab specimen.

For self-testing *in vitro* diagnostic use.

**TESTING PROCEDURE**

**Package Insert**  
REF: ICOV-502S  
For Self-testing | English

**READ RESULTS**  
Please share your test result with your healthcare provider and carefully follow your local COVID guidelines/requirements.

**POSITIVE:** You can see two colored lines. One colored line appears in the control line area (C), and the other line appears in the test line area (T). Note: The intensity of the color in the test line region (T) will vary based on the amount of SARS-CoV-2 antigen present in the sample. So any shade of color in the test region (T) should be considered positive. The positive result means that you are most likely to suffer from COVID-19, but the results should be confirmed. Carefully follow your local COVID guidelines / requirements. If it is necessary, your doctor may prescribe PCR detection reagent for confirmation.

**NEGATIVE:** You can see only one colored line appears in the control line area (C). There is no line appears in the test line area (T). This means you are unlikely to have COVID-19. However, it is possible for this test to give a negative result that is incorrect (a false negative) in some people with COVID-19. This means you could possibly still have COVID-19 even though the test is negative. In addition, it is recommended to repeat the test in 1-2 days, because the coronavirus cannot be accurately detected at all stages of the infection. Even with a negative test result, distance and hygiene rules must be observed, migration/traveling, attending events and etc should follow your local COVID guidelines/requirements.

**INVALID:** There is no line appears in the control line area (C). The most likely reason for this situation is insufficient specimen size or incorrect process operation. It is recommended to review the procedure, and then re-operate with a new test. Stop using the test kit if the problem is not resolved, and contact your local COVID-19 Center.

**INTENDED USE**  
The COVID-19 Antigen Rapid Test (Swab) is a single-use test kit intended to detect the novel coronavirus SARS-CoV-2 that causes COVID-19 in self-collected nasal swab specimen from symptomatic / asymptomatic individuals who are suspected of being infected with COVID-19. This test is designed for the use by layperson.

A positive result indicates the presence of SARS-CoV-2. Individuals whose test results are positive should self-isolate and seek help from relevant healthcare institutions. A positive result may also be caused by bacterial infection or co-infection with other viruses. A negative result may also be infected with SARS-CoV-2. Individuals whose test results are negative continue to experience COVID-like symptoms should seek follow-up help from relevant healthcare institutions.

The COVID-19 Antigen Rapid Test (Swab) only indicates a preliminary result. The confirmation of the final result should be based on the clinical diagnosis.

**BACKGROUND**  
The novel coronaviruses belong to the *Beta* genus. COVID-19 is an acute respiratory infectious disease. People are generally susceptible. Currently, the patient infected by the novel coronavirus is the main source of infection; asymptomatic infected people can also be an infectious source. Based on the current epidemiological investigation, the incubation period is 1 to 14 days, mostly 3 to 7 days. The main manifestations include fever, fatigue and dry cough. Nasal congestion, runny nose, sore throat, myalgia and diarrhea are found in a few cases.

**TEST PRINCIPLE**  
The COVID-19 Antigen Rapid Test (Swab) is a qualitative membrane-based immunoassay for the detection of SARS-CoV-2 Nucleocapsid protein Antigens in human nasal swab specimen.

**REAGENTS**  
The test contains monoclonal anti-SARS-CoV-2 antibody and goat anti-mouse IgG as the capture reagent and monoclonal anti-SARS-CoV-2 antibody and mouse IgG as the detection reagent.

**MATERIALS**  
Materials Provided

- Test cassette
- Extraction buffer
- Package insert
- Sterile swab
- Biosafety Bag (optional)
- Tube holder

Materials required but not provided

- Timer

**LIMITATIONS**

1. Failure to follow these procedures may affect test performance.
2. The COVID-19 Antigen Rapid Test (Swab) only indicates the presence of SARS-CoV-2 antigens in the specimen.
3. Negative results cannot rule out SARS-CoV-2 infections, especially in those who have been exposed to the virus. If the test result is negative or non-reactive and clinical symptoms persist, it is because they are very early infection virus may not be detected. It is recommended to test again with a new kit or test with a molecular diagnostic device to rule out infection in these individuals.
4. Positive results of COVID-19 may be due to infection with non-SARS-CoV-2 coronavirus strains or other interference factors.
5. If the virus concentration in the specimen is too low, false negative results may occur.

**PERFORMANCE**  
Clinical performance

A clinical evaluation was conducted comparing the results obtained using the COVID-19 Antigen Rapid Test with RT-PCR (nasopharyngeal swab) test result. The clinical trial included 997 nasal specimens. The results demonstrated 97.0% sensitivity and 99.9% specificity with an overall accuracy of 98.0%.

**EXTRA INFORMATION**

1. How do I know if the Test worked well?

The COVID-19 Antigen Rapid Test is a rapid chromatographic immunoassay for the qualitative detection of SARS-CoV-2 antigens present in self-collected swab specimens.

**INTESTED USE**  
When the control line(C) appears, it means the test unit is performing well. 2. How soon can I read my results? You can read your results after 15 minutes as long as a colored line has appeared next to the Control region(C), do not read result after 20 minutes.

3. When is the best time to run the test?

Test can be done at any time of the day.

4. Can the test be wrong? Are there any factors that can affect the test result?

The result will only give accurate results as the fresh human nasal swab is used and follow the instructions carefully. Nevertheless, the result can be incorrect. Non-SARS-CoV-2 coronavirus strains or other interference factors may cause a preliminary result.

5. How to read the test if the color and the intensity of the lines are different?

The color and intensity of the lines have no importance for result interpretation. The test should be considered as Positive whatever the color intensity of the test line (T).

6. What do I have to do if the result is positive?

A positive result means the presence of SARS-CoV-2 antigens. A positive result means it is very likely you have COVID-19 and the result should be confirmed. Immediately go into self-isolation in accordance with the local guidelines and immediately contact your general practitioner / doctor or the local health department in accordance with the instructions of your local authorities. Your test result will be checked by a PCR confirmation test and you will be explained the next steps.

7. What do I have to do if the result is negative?

A negative result means that you are not positive or that the viral load is too low to be detected. You can repeat the test if you feel to give a negative result that is incorrect (a false negative) in some people with COVID-19. This means you could possibly still have COVID-19 even though the test is negative. In addition, you can repeat the test with a new kit. In case of suspicion, repeat the test after 1-2 days, as the coronavirus cannot be precisely detected in all phases of an infection. Even with a negative test result, distance and hygiene rules must be observed, mitigation/traveling, attending events and etc should follow your local COVID guidelines/requirements.

**LITERATURE REFERENCES**

1. BACKINGER, C.L. and KINGSLEY, P.A., Recommendation for Developing User Instruction Manuals for Medical Devices Used in Home Health Care, Rockville, MD, U.S. Food and Drug Administration, Center for Devices and Radiological Health, HHS Pub. FDA 93-258.

**INDEX OF SYMBOLS**

	In vitro diagnostic medical device		CE mark
	Temperature limit 2-30°C		Tests per kit
	Use by		Batch code
	Manufacture		Consult Instructions For Use
	Authorized Representative in EU		Catalog #
	Do not reuse		CE mark
	Keep dry		

**CITEST DIAGNOSTICS INC.**  
170-422 Richards Street  
Vancouver BC, V6B 224, Canada

**CMC MEDICAL DEVICES & DRUGS, S.L.**  
C/ Horacio Lengo N° 18, CP 29006, Málaga-Spain

Statement: Information about manufacturer of Sterile swab is placed on the packaging.

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